

INSTRUCTIONS FOR USE



PROTECTIVE PROPERTIES

Our mats are made with 100% SmartTech polyurethane technology which is our ground-breaking and proprietary engineered material system. The mats have been tested for how well they can absorb shock and reduce impact against ASTM F355 standards using Head Injury Criteria. Features of our mats include:

- One-piece construction which ensures the mats will not come apart, bubble up, or delaminate with prolonged use
- Able to withstand many impacts over time and still exceed impact standards
- Able to resist water and toxins from being absorbed into the mat (studies show 99.9% population reduction of staph aureus bacteria over 24 hours)
- Able to resist slippage



USER INSTRUCTIONS

1. Place the mat lengthwise/parallel next to the bed. If using a bed with siderails, lower the bottom rail to encourage the patient/resident to exit on the matted side.
2. When used with a chair, position the mat right in front of the chair. Place it as close to the chair as possible where the chair can still function as normal.



⚠ WARNING

Mats should always be placed on a clean, dry floor. Any liquids on the floor near, on or underneath the mat should be cleaned and dried right away. Even small amounts of liquid under the mat can cause the mat to slip, leading to accidental falls. It is NOT recommended to use these mats in spaces that are constantly wet or oily.

Staff, caregivers, and patients/residents should be told of the use of the mat and should take care when stepping on and off the mat to avoid tripping.

STORAGE AND HANDLING

The mats are meant for indoor use and may be stored at normal room temperature. Do not store the mats in very humid areas.

The mats should be stored flat and can be stacked on one another to save space when not in use. Do not store more than 10 mats on top of each other.

The mats can be moved by picking them up from one side or by rolling them and carrying them from both sides at the same time.

NOTE: Due to the random possibility of falls, Prevention Plus Performance Products makes no guarantee, express or implied, that the user is protected from trauma to the head or body.

CLEANING INSTRUCTIONS

Cleaning the mats is easy with soap and water and commonly used cleaning products. Mats can be wiped clean or may be cleaned with flat floor mops.

⚠ CAUTION

DO NOT USE ACID, ACETONE OR UNDILUTED BLEACH BASED CLEANING PRODUCTS.

⚠ WARNING

Ensure the top and bottom of the mats are completely dry before placing them next to the bed or chair.

DISPOSAL

Clean mats prior to disposal per facility policy.

⚠ CAUTION

Dispose of mats per facility policy.